

Taking nasal swabs

Take the right samples to get best results

 <p>1</p>	<p>Nasal swab sets</p> <p>A Set for 20 piglets: 20 small nasal swabs and Virocult®.</p> <p>B Set for 20 animals of all age groups (see A, but with large swabs).</p>	 <p>2</p>	<p>One nasal swab per animal.</p> <p>Insert nasal swab in both nostrils.</p> <p>Make sure to insert the nasal swab deeply into each nostril.</p>	 <p>3</p>	<p>Place the swab in the tube and cut the plastic stick slightly behind the viscose head.</p> <p>Do not leave the stick too long, because the Virocult® will not close properly.</p>
 <p>4</p>	<p>Label the tube using a waterproof pen.</p> <p>For example: animal numbers, sampling date, stock, type of pen, special characteristics ...</p> <p>In your own interest, please send a preliminary report with the samples.</p>	 <p>5</p>	<p>It is essential to ship the samples at the beginning of the week with a frozen cooling element.</p> <p>For samples at the end of the week, store nasal swabs in the refrigerator (collection of several swabs possible; do NOT freeze!).</p> <p>Always ship nasal swabs with cooling elements.</p>	<p>To ship the nasal swabs please use the form "Submission form for isolation of influenza viruses".</p> <p>Ask your local sales representatives for a free nasal swab set.</p>	<p>Content of diagnostic KIT:</p> <ul style="list-style-type: none">1 × cooling element1 × safety bag1 × UN-sticker1 × address sticker20 × small or large nasal swabs20 × medium tubes (Virocult®)absorbable tissue



If you have any questions, please contact your IDT Technical Service Manager:

Dr. Verena Gotter 0049 - 152 018 275 81

There is no obligation to purchase IDT products in connection with, or after, the use of the IDT diagnostic tests.

